Corey's Medium - Tissue Softener Plus

HCT-S02-453

EN

Solution intended to be used to soften hard mineralised tissues such as bone, nail, cartilage and haemorrhagic samples. This product ensures optimal tissue integrity without compromising the binding site of epitopes, ideal for immunohistochemistry procedures.

Shake well to activate ready to use solution.

For laboratory use.

Please see expiry on product label.

Components: contains ethyl alcohol.

Instructions for Use

1. Expose the full face of the paraffin block.
2. Immerse in Corey's Medium for 30 minutes at room temperature. (This time can be extended for more difficult specimens). To de-wrinkle haemorrhagic tissue samples immerse for 10 minutes.
3. Remove tissue blocks from solution and wipe excess softener from the surface.
4. Place tissue block on a cold plate for 3-5 minutes.
5. Section using a standard microtome.
6. Discard spent Corey's Medium solution.

Note: The solution cannot be reused. Once dispensed from the container the liquid should be discarded after use.

Produced and Manufactured
HistoCare GSS Ltd
www.histocare.com
0203-488-3706 

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